

## How to LOAD/REMOVE a syringe

***You will never need to use force to load or remove a syringe.***

If the syringe does not easily load or can not easily be removed from the pump, STOP. Make sure the pump is in the off position, and that the black tab is at the end of its track. If the black tab is not at the end of its track, wind the large knob clockwise and try again. If the black tab is always wound back the full length of its track, you will never have a problem loading or removing a syringe.



### Step 1

Make sure that the pump is off and that the black tab inside the clear view syringe shield is at the end of its track. If the black tab is not at the end of its track, wind the large knob clockwise.



### Step 2

With syringe gradations facing up, load the syringe and its tubing into the pump. (Make sure the luer disc is fully seated in the pumps nose.)



### Step 3

After the infusion is complete, turn the pump off and wind the large knob clockwise until the black tab is at the end of its track. Remove the empty syringe by gently pushing it back and up out of the pump.

***You will never need to use force to load or remove a syringe.***