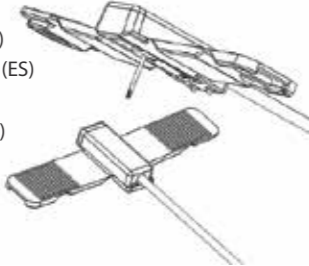
Subcutaneous Safety Needle Set

HighFlo
Subcutaneous Safety Needle Sets

- Subcutaneous Safety Needle Set (EN)
- Souprava subkutánních bezpečnostních jehel (CS)
- Subkutant sikkerhedskanylesæt (DA)
- Set mit subkutanen Sicherheitskanülen (DE)
- Juego de agujas de seguridad subcutáneas (ES)
- Ihonalainen turvaneulapakkaus (FI)
- Kit d'aiguilles de sécurité sous-cutanées (FR)
- ערכת מחט בטיחות תת-עורית (HE)
- Szubkután biztonsági tűkészlet (HU)
- Set di aghi di sicurezza sottocutanei (IT)
- Subkutant sikkerhetskanylesett (NO)
- Набор безопасных игл для подкожных инъекций (RU)
- Infusionsset med säkerhetskanyl (SV)

MADE IN 
RMS MEDICAL PRODUCTS
Manufacturer
RMS Medical Products
24 Carpenter Rd.
Chester, NY 10918 USA
800-624-9600 toll free
845-469-2042 local
rmsmedicalproducts.com
CE 0086
EC REP European Representative
RMS Medical Products UK, Ltd.
146 Allerton Rd.
Mossley Hill, Liverpool
L18 2DH, UK

HighFlo Subcutaneous Safety Needle Sets™ are a registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2016 RMS Medical Products; All Rights Reserved. instructions_needle_sets (342102)RevE

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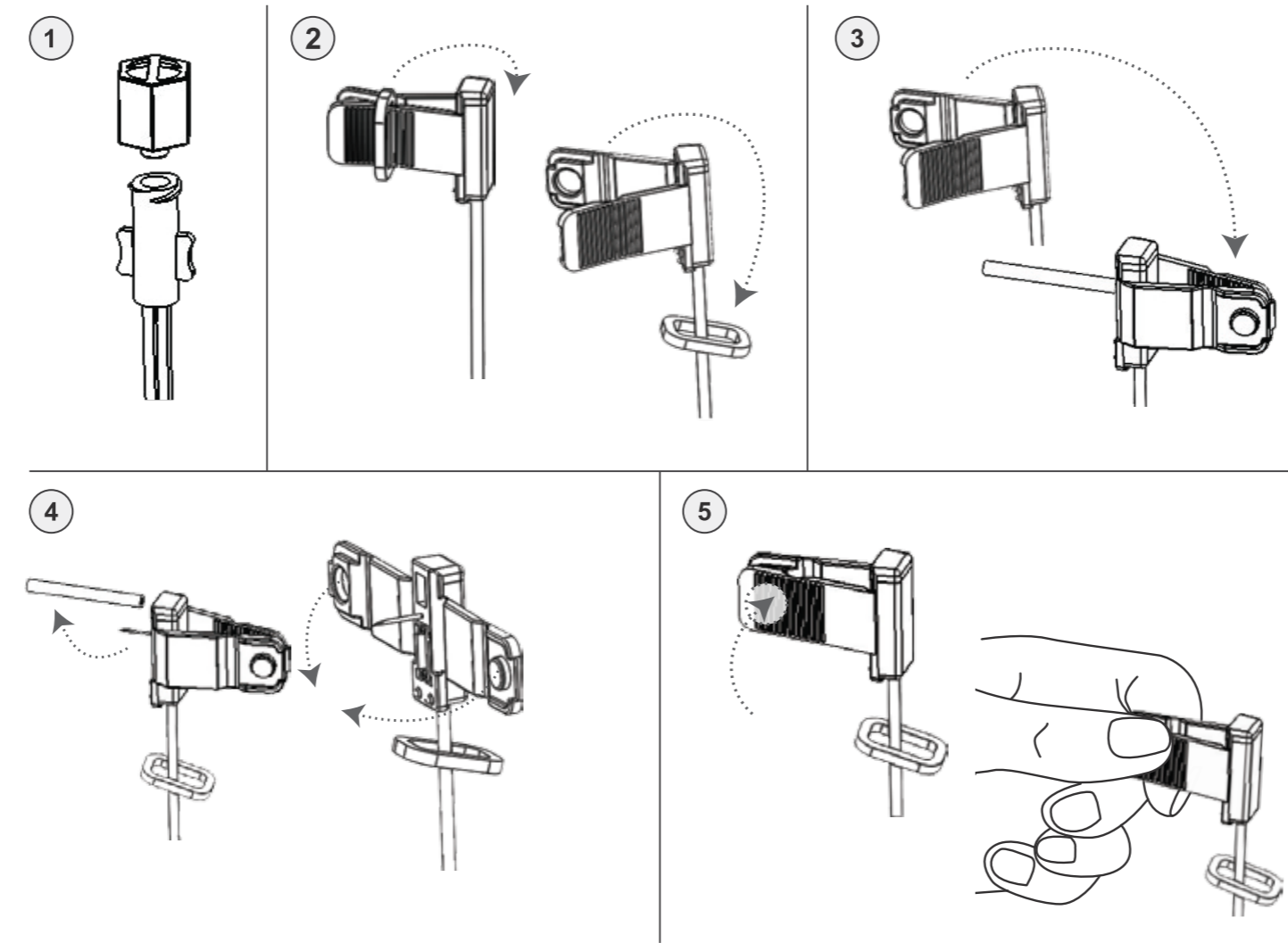
EN

Indications for use:

RMS Subcutaneous Needle Sets are intended for the subcutaneous infusion of prescribed liquid medicines from an external infusion pump or syringe.

Caution:

- U.S. Federal Law restricts this device to sale by or on the order of a physician.
- Use subcutaneous needle sets only for the patient in which the device is prescribed and only for its intended use.
- Subcutaneous needle sets are only sterile and non-pyrogenic if the packaging is unopened and undamaged. Do not use a needle set from an open or damaged package.
- Subcutaneous needle sets are single use items which should be disposed of after use. Please contact your healthcare provider for available sharps disposal containers.
- Do not clean or re-sterilize needle sets.
- Be sure that the needle guard is removed before insertion.
- Assure that the needle is not bent prior to insertion.
- Never prime the needle set or attempt to free a clogged tube while the needle is inserted. You may accidentally inject too much medication.
- When infusing through multiple needles, over infusion may occur to remaining sites if one or more of the needles are occluded.
- Inaccurate medication delivery, infection and/or site irritation may result from improper needle insertion and maintenance of the infusion site.
- Check the infusion set periodically to ensure that the needle remains firmly in place. If the needle is not in place, remove it and use a new needle set.
- When using needle sets for the first time, do so in the presence of a healthcare provider.

Instructions for use:

1. Remove the end cap from the female luer and connect the luer to the source of infusion.
2. Remove the wings closure band to separate each wing.
3. Bring the wings together and hold them between two fingers so that the needle guard is exposed.
4. Remove the needle guard and insert each needle into the subcutaneous tissue; cover with adhesive dressing.
5. When complete, remove adhesive and pull each needle out by bringing the wings together. Snap closed to capture the needle and discard.

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