


get to know FREEDOM60[®]

Syringe Infusion System


Easy to use & learn

Only 2 controls. Just wind, load & go!

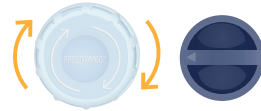


Safe constant pressure

Highly accurate constant pressure pump for safe delivery. Maintains pressure of 13.5 psi.

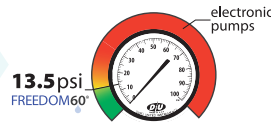

Advanced technology

All parts are designed to fit & work together within the pump. The Negator motor is the 'heart' of the FREEDOM60[®].

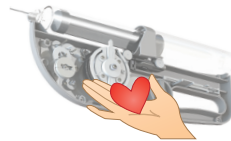

Quality assured

Made in the USA with the highest level of quality control for defect-free pumps. No product maintenance needed.



DynEQ[™] (Dynamic Equilibrium)

If the pressure in your sites increases, the system will immediately respond by automatically slowing down.




Portable

No batteries or electricity needed. Comes with a travel pouch to promote patient independence.




Standard 60ml capacity

Easy to handle and purge of air. Can be filled from 1ml to 60ml.

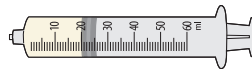

Rugged

Robust design with a molded ABS plastic housing. No sensitive electric parts to get jumbled.

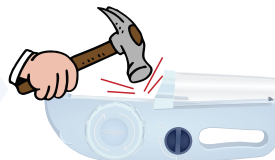

Spring-loaded syringe holder

Locks the syringe into the pump & maintains proper syringe orientation, prevents inadvertent removal during administration.



Recessed on/off switch

No inadvertent shut off.



Large wind knob

Ergonomic, easy to grasp & turn, disengages when the pump is ON, cannot be over-wound.



Syringe shield

Easy to check infusion progress, protects the syringe.




precision FLOW RATE TUBING[™]

RMS Precision Flow Rate Tubing[™] controls the infusion rate in the FREEDOM60[®] system.




Luer disc

Helps prevent contamination. For patient safety, the pump is designed not to work unless a set with the disc is present.

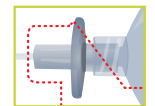

Deliver in minutes to days

Sets from F0.5 to F2400 offer precise infusion rates for a variety of administration times.

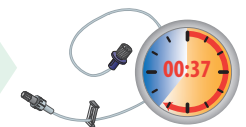

Emergency slide clamp

Stop flow immediately, if needed. F-number printed on clamp to assure correct tubing is being used.



Low residual volume

Consistent delivery of medication for a full therapeutic dose without expensive drug waste, $\leq 0.16\text{ml}$.



The FREEDOM60[®] Syringe Infusion System, RMS Precision Flow Rate Tubing[™] and DynEQ[™] are registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2016 RMS Medical Products; All Rights Reserved. RHL_INFG_F60benefits_03-10005vA

rmsmedicalproducts.com 845-469-2042 800-624-9600

